


CPI Integrated DAP

The Magic of Integration: DAP Wizard

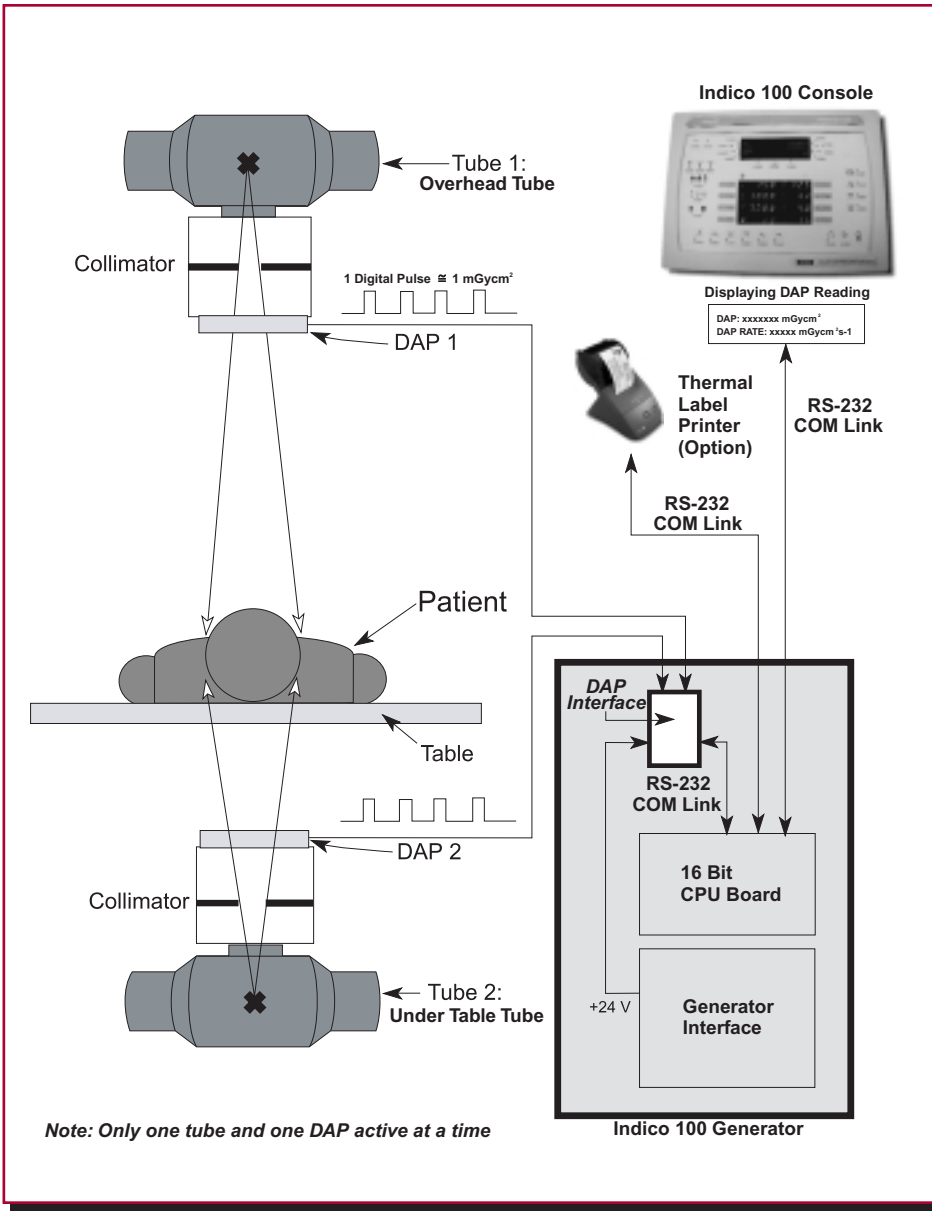

Easy Setup and Calibration Facilitated by Genware

$$(D) + (A) = (P)$$

Measure X-ray ("Dose") at a calibrated distance.


Measure X-ray field size ("Area") at distance calibration exposure(s) were made

Press ENTER ... DAP Wizard automatically calibrates DAP!

Enter exposure dose reading from calibrated radiation probe into DAP Wizard.

Enter field size area from exposure film into DAP Wizard.

NO potentiometers to adjust!
NO calculations!

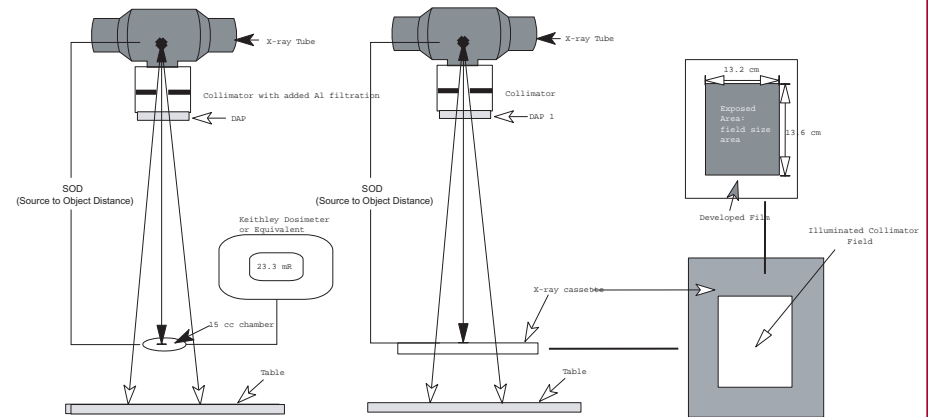

$$\begin{matrix} (D) & + & (A) \\ \text{Calibrated Exposure} & & \text{Field Size Area} \\ \text{Dose Reading} & & \\ (P) & & \\ \text{GENWARE} & & \\ \text{DAP WIZARD} & & \\ = & & \\ \text{Automated DAP Calibration} & & \end{matrix}$$

All unit conversion and calculations performed by CPI DAP Wizard...

DAP interface electronically (and remotely) calibrated! There are no potentiometers at the collimator to adjust...

Simple - Fast - Accurate